

- Biocompatible-made of all natural NOCC chitosan polymers
- Minimizes bleeding and edema after surgery
- Easily conforms to nasal anatomy
- Offers easier placement with a superior field of view
- No special storage or refrigeration required

PosiSep® is indicated for use as a hemostat dressing/intranasal splint intended to minimize bleeding and edema and to prevent adhesions between the septum and the nasal cavity. PosiSep® is constructed from proprietary NOCC chitosan polymers into a patient-comfortable sponge which is placed in the nasal cavity after surgery or trauma. After hydration, PosiSep® will immediately expand and turn into a gel, then begin clearing the sinus cavity. Subsequent irrigation will cause the product to dissolve, thus optimizing patient tolerance and comfort. Typical clearance time is about 3-4 days.

- Dissolves naturally by way of gentle irrigation and aspiration
- Easy to apply-no mixing or additional prep time necessary
- Widely studied and published anti-adhesion properties of our proprietary NOCC (N,O-carboxymethyl chitosan)

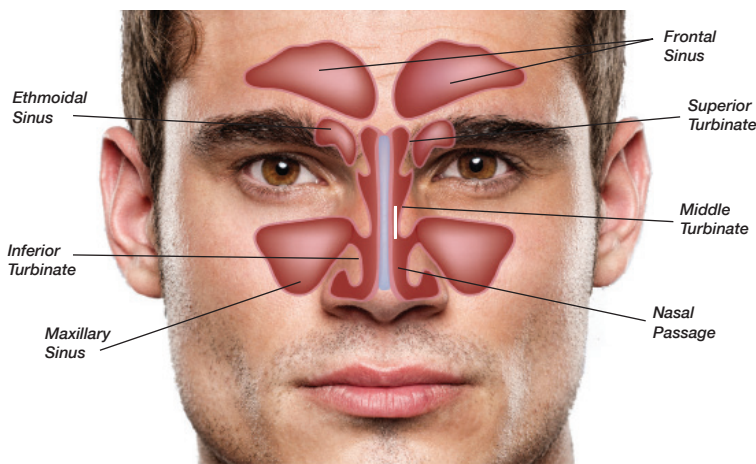


diagram not to scale

PosiSep® is a unique NOCC-based hemostat dressing/intranasal splint that has been custom formulated for OR and office-based FESS procedures with minimal bleeding and where rapid clearance of the product is desirable. PosiSep® can be cut into smaller sections for optimal placement.

Before Hydration



After Hydration



When PosiSep® is hydrated, its smooth gel action separates sinus mucosal tissue.

What is NOCC?

NOCC is a proprietary medical grade, N,O-carboxymethyl chitosan polymer manufactured by Hemostasis. It is made from chitin, an amino-polysaccharide removed from the powdered shells of crustaceans. NOCC has unique properties which make it advantageous for use in medical applications.

CHITOSAN TECHNOLOGY

ORDERING INFORMATION PosiSep® is available in two sizes. See ordering information below.

Part Number: Description:

9210554 PosiSep® Hemostat Dressing/Intranasal Splint 0.5" x 1.5" 5pk (US)
 9210574 PosiSep® Hemostat Dressing/Intranasal Splint 0.6" x 2.0" 5pk (US)

How Supplied:

PosiSep® is packaged five (5) sterile, single use units per carton. One (1) sterile unit contains one (1) Hemostat Dressing/Intranasal Splint. PosiSep® is terminally sterilized by gamma irradiation.

CAUTION: Federal (U.S.) law restricts this device to sale by or on the order of a physician. PosiSep® is contraindicated for patients who have known allergies to shellfish. For detailed information regarding indications for use, warnings and precautions, see Instructions for Use.

To place an order or for more information on PosiSep® or our other medical technologies, please contact a sales representative by calling 866-612-2568.