



Designed specifically for providing localized drug delivery in the sinus frontal ostia.

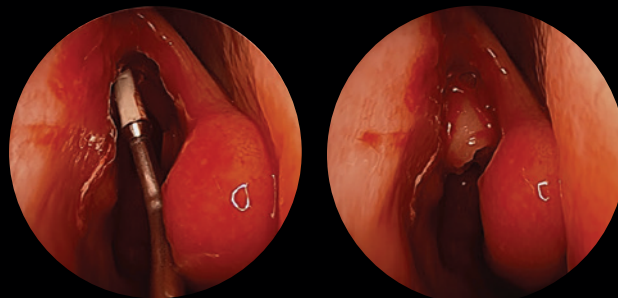

VECTRA™ puts localized drug delivery in the hands of surgeons.

VECTRA™ was specifically developed to administer medicinal substances to treat nasal disorders. The patented VECTRA™ microsponge is supplied in a compressed tubular design preloaded on one end of an insertion tool. The opposite end of the tool has a Luer fitting which is connected to a syringe containing the medicine to be delivered. The tool is used to place the VECTRA™ microsponge in the frontal ostia. Once in place, the medicine can be delivered to the VECTRA™ device. As the medicine hydrates the VECTRA™ microsponge, the microsponge expands to conform to the anatomy of the ostia. The tool is then removed and the VECTRA™ microsponge remains in place. The intimate contact allows for localized drug delivery to the desired target area.

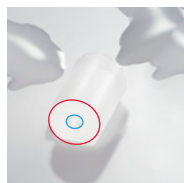


VECTRA™ 7 (Hydrated)

- Outer Diameter **7mm**
- Lumen Diameter **3.5mm**
- Length **15mm**



VECTRA™ 15 (Hydrated)

- Outer Diameter **15mm**
- Lumen Diameter **3.5mm**
- Length **20mm**

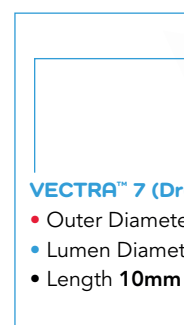

VECTRA™ 7 (Dry)

- Outer Diameter **2mm**
- Lumen Diameter **1mm**
- Length **10mm**

VECTRA™ 15 (Dry)

- Outer Diameter **6mm**
- Lumen Diameter **1mm**
- Length **18mm**

Safe. Simple. Effective.

VECTRA™. Designed in conjunction with sinus surgeons.

- Designed for localized drug delivery
- Easy-to-use
- Easily placed and hydrated in the frontal ostia in both operating room and clinic
- Insertion tool provides easy, accurate placement; and drug delivery to microsponge
- Patented tubular design maintains an open lumen between the sinus and nasal cavity
- Dissolves away in 3-6 days
- No special storage or refrigeration required
- Affordable

Ordering Information

Part Number:	Description:	How Supplied:
9210864	VECTRA™ F7	VECTRA™ F drug delivery device is packaged with two (2) sterile, single use units per box. One (1) sterile unit contains one (1) VECTRA™ microsponge and one (1) frontal sinus insertion tool. VECTRA™ is terminally sterilized by gamma irradiation
9210894	VECTRA™ F15	

CAUTION: Federal (U.S) law restricts this device to sale by or on the order of a physician. For detailed information regarding indications for use, a list of medicines evaluated for suitability with Vectra, warnings and precautions, see instructions for use.



To place an order or for more information on VECTRA™ F or our other medical technologies, please contact a sales representative by calling our toll free number 866-612-2568